



STEP 1:

Unzip downloaded
folder in your computer.



STEP 2:

Select the features
you want to install.



STEP 3:

You will find 2 different folders **inside the main folder**.



1. System name (EN)
1. Nombre del sistema (ES)



2. System name + Replica (EN)
2. Nombre del sistema + Replica (ES)

STEP 4:

Find the folder named **DentralCADApp** in your computer:



Copy in:

.../DentalCADApp/**library/implant**

The folder named number 1. (Nombre del sistema)

Copy in:

.../DentalCADApp/library/**modelcreator/Implants**

The folder named number 2. (Nombre del sistema + Replica)